

THOROUGH *Topicalization*

PRECISE *Anesthetic Placement*

Soft Tissue Retractor with built-in atomizer, facilitates anesthetic application to the difficult-to-reach glottic opening.

- Thorough and precise anesthetic deposition – improves patient comfort (no coughing/gagging)
- True awake intubation – minimal use of sedatives and narcotics
- Saves time/less stress – faster throughput
- Exact dosing – less risk of toxicity
- Prevention of cross-contamination



WOLFE TORY
MEDICAL, INC.

Elegant solutions for complex issues.

The logo for MADgicWand, featuring the brand name in a blue, stylized font with a star above the 'i' in 'gic'. Below the name is the text 'PHARYNGEAL TISSUE RETRACTOR/ATOMIZER' in a smaller, blue, sans-serif font.

MADgicWand™
PHARYNGEAL TISSUE RETRACTOR/ATOMIZER

THE MADgicWAND™ PHARYNGEAL TISSUE RETRACTOR/ATOMIZER: THOROUGH TOPICALIZATION, PRECISE ANESTHETIC PLACEMENT

Step 1:

Anesthetize the mouth and oropharynx with the MADgicWand™.

Allow time for the anesthetic to work

Step 3:

With the MADgicWand™ aimed at the vocal cords, ask patient to take slow, deep inspirations and atomize 0.2-0.4cc volumes with each breath to anesthetize cords and upper larynx.



Step 2:

Incrementally spray posterior tongue and throat with topical anesthetic, slowly advancing the MADgicWand™ as tolerated.



Step 4:

Proceed with intubation.



Applications:

- *Difficult Intubation*
- *Cervical Spine Injury Intubation*
- *Bronchoscopy*

ORDERING INFORMATION:

Reorder Number	Quantity / Case	Description
MAD650	20	MADgicWand™ Mucosal Atomization Device for atomizing topical solutions. With 5 mL syringe

SPECIFICATIONS

Typical particle size:	System dead space:
30-100 microns	0.25 mL



Wolfe Tory Medical, Inc.
79 West 4500 South, Suite 18
Salt Lake City, Utah 84107 USA
Phone: 801-281-3000 Fax: 801-281-0708
Toll Free: 1-888-380-9808
www.wolfetory.com wolfetory@wolfetory.com

