

Patient discomfort (coughing, gagging, bucking)
causing delays, adding stress?

Get ***One*** Hand on the Problem


NEW

WT

WOLFE TORY
MEDICAL, INC.

Elegant solutions for complex issues.

Simplified
Fiber Optic
Intubation

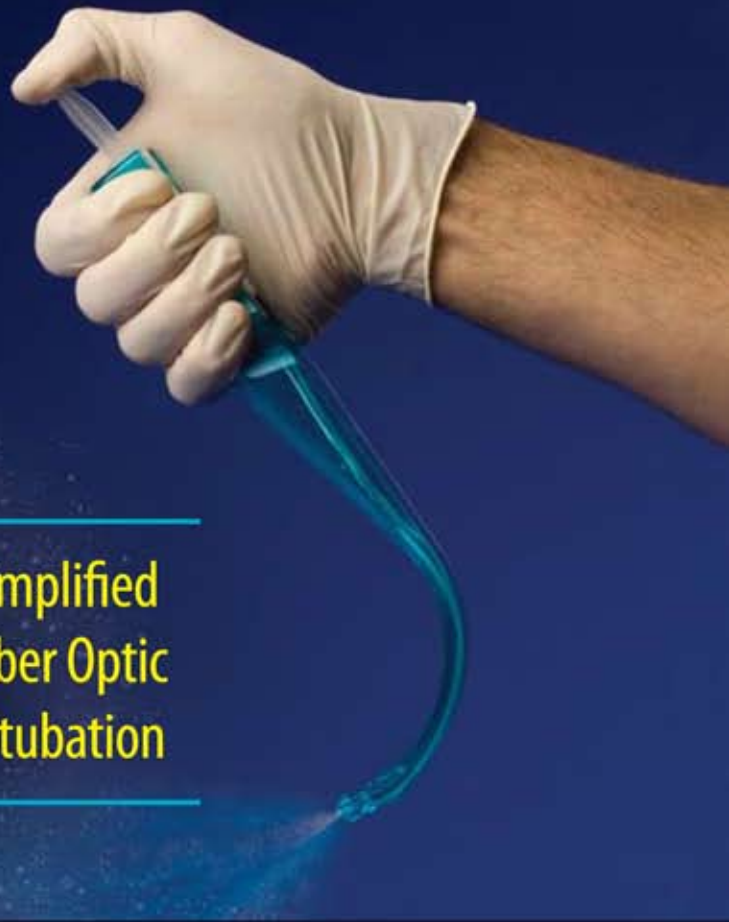

THOROUGH *Topicalization*

PRECISE *Anesthetic Placement Prior
to Fiber Optic Intubation*



Now everyone can relax.

WT

WOLFE TORY
MEDICAL, INC.

Elegant solutions for complex issues.

MADgicWand™
PHARYNGEAL TISSUE RETRACTOR/ATOMIZER

On your next fiber
optic intubation...

WAVE AWAY THE STRESS

with the **MADgicWand™**

*Precisely Delivered
Laryngeal Anesthesia,
Improved Patient Comfort.*

**Rigid blade with built-in atomizer retracts
tissue, facilitates anesthesia application
to the difficult-to-reach glottic opening.**

- *Well tolerated*
- *Minimize patient injury*
- *Optimize patient comfort*
- *Easy, one handed operation*
- *Exact dosing*
- *Less stress*



MADgicWand is a trademark of Wolfe Tory Medical, Inc. © 2008 Wolfe Tory Medical, Inc. All rights reserved. Made in the U.S.A.



Step 1:

First atomize topical anesthetic onto the anterior, then posterior aspect of mouth; wait a few minutes.

Step 2:

Insert the blade of the MADgicWand™ along the floor of the mouth with tip directed towards the laryngeal inlet. Retract the tongue and spray anesthetic in short bursts while patient takes slow, deep inspirations. Wait a few minutes, then proceed with intubation.

ORDERING INFORMATION:

Reorder Number	Quantity / Case	Description
MAD650	20	MADgicWand™ Mucosal Atomization Device for atomizing topical solutions. With 5 mL syringe
MAD680	10	MADgicWand™ Fiber Optic Intubation kit for applying airway anesthesia. Includes MADgicWand retractor with 5mL syringe, intubating airway with oxygen/atomization channel, pediatric-sized MADgic atomizer with 3mL syringe, angled applicator sponges and lubricant jelly

SPECIFICATIONS

Typical particle size:	System dead space:
30-100 microns	0.25 mL



Wolfe Tory Medical, Inc.

79 West 4500 South, Suite 18

Salt Lake City, Utah 84107 USA

Phone: 801-281-3000 Fax: 801-281-0708

Toll Free: 1-888-380-9808

www.wolfe-tory.com wolfe-tory@wolfe-tory.com

P/N 30-0220 Rev 5/08

**PHARYNGEAL TISSUE
RETRACTOR/ATOMIZER**